

Dog Cart and Pet Wheelchair comparison page

Contributed by Administrator
Sunday, 04 March 2007
Last Updated Monday, 28 April 2008

Dog Cart and Pet Wheelchair comparison page

This is the 'Comparing Dog Carts page.' You can review the different kinds of dog carts.

The Key to Cart Pages on the this site is an index with links to all of the cart pages on this site.

When you're done with this page, our "Ordering Page" has pricing information and the measurement charts you'll need. Remember, if you get lost, just go to the Key to Cart Pages or back to <http://www.DogKarts.com>.

Support Sling Dog Cart

K9 contoured leg ring Dog Cart

Major cart manufacturers offer these proven cart designs. For the first step in your decision process, understand the difference between a Contouring Leg Ring Cart and a Support Sling Dog Wheelchair. Become familiar with the return policy and some of the options available.

A Support Sling dog cart is assembled from lengths of sturdy tubing and connected with locknut-equipped bolts. A soft, gender-specific saddle support sling is worn by the animal and clips securely to the frame. See "Wearing" below for details

A K9 contouring leg ring Dog Cart is custom assembled for the animal with a lightweight aluminum tubing and custom molded plastic connectors. Foam-encased cables support the animal's pelvis. Hinged supports allow the cart to be folded for easy transportation. See "Wearing" below for details

Price: \$220 - \$450

Price: \$275 - \$550

Delivery: Dog Carts are shipped in 8 to 10 Business Days. Some simple assembly required on larger wheelchairs.

Delivery: 3 to 5 Business Days to build. Assembly for large carts involves unfolding, inserting clips and pins.

Return Policy: Dog Carts may be returned within 30 days if not satisfied for a 70% refund. dog cart warranty.

Return Policy: Dog Carts may be returned within 20 days for 65% refund. Details of dog cart warranty

Wearing: Pets wears a comfortable harness that connects to front of wheelchair. The harness is put on the pet first, then connected to the cart. Dog Cart Construction

Wearing: Animals back legs are lifted into the contouring leg rings made of foam coated cables. A rear support leash or back harness can be used to help lift the dog. Dog Cart Construction

These dog carts have received excellent reviews from veterinarians, users and their pets. Each has their advantages and disadvantages. To speak with other cart users, go to Our Discussion Board.

Business: Pets can urinate and defecate in the sling. From time to time the sling get soiled and can be washed easily.

Business: Pets can urinate and defecate in the without any difficulty as they run along. All of the leg ring pelvic supports are made exactly the same and work perfectly on both genders.

Pet Growth: Adjustable weight balance is used to meet the changing needs of your pet- whether your pet gains more weight from regaining muscle mass, or loses weight.

Pet Growth: If your pet outgrows the first cart, return it to us with photos of your pet in the cart and we will replace it at a reduced cost giving you credit for the parts we can still use or recycle.

Dogs cannot lay down in a cart unless very short legged. Often, they'll prop themselves on a pillow. See Photos of dogs laying down in Dog Carts .

Training wheels and front end extensions can be added to a rear wheel cart to offer front leg support. details.

Front Supportive Extensions are available for pets with balance issues due to stroke or weak front legs due to advanced degenerative myelopathy, arthritis, elbow dysplasia. details

Support Sling Dog Cart

K9 Contouring Leg Ring Dog Cart

Front Wheel Extensions available for dogs with weak front legs.

Front wheel dog carts support animals with front leg problems.

Stirrups are included with every wheelchair and allow for full range of motion for pets with residual movement in hind limbs.

Leg Slings are included with the cart. If the animal's rear legs drag, they can be held off the ground with these comfortable slings.

Amputees: Adjustable weight balance and custom modifications are done for amputees as well as for pets that have unique disabilities; double amputations, birth defects, large tumors etc.

Amputees: Counter-weights available for amputees. details

Support Sling quad carts offer four-wheel support. details.

Four wheel dog carts are available. details

Adjust: Height and Length is adjustable to meet pet's preferences and needs.

Adjust: The carts are adjustable 4 inches total in height and length on average with the smaller carts being a little less and the larger carts being a little more.

Your pet can easily relieve itself and perform its normal bodily functions while in any of the carts.

Carts available for pets that weigh up to 250 + pounds.

Carts are made for pets weighing 275+ pounds

Once you understand some of the differences and options, our "Ordering Page" has pricing information and the measurement charts you'll need. Remember, if you get lost, just go to the Key to Cart Pages or back to <http://www.DogKarts.com>.